



MTI, Inc.  
 3655 Ninigret Drive  
 Salt Lake City, UT 84104  
 US & Canada Sales (800) 924-4655  
 International Sales (801) 875-4999  
 Fax (801) 952-0548  
 www.mti.net

## MTI Suggested Medical Disinfectants Policy

MTI's standard policy on disinfecting MTI medical equipment is to use a 10% sodium hypochlorite (bleach) and 90% water solution. If stronger or other disinfecting and cleaning solutions are used, the surface is at risk to be damaged or possibly cause discoloring, cracking, or other types of damage to the surface.

MTI understands that many customers prefer to use a premixed disinfecting solution in a spray or wipe. It is MTI's recommendation not to use sprays because over time the overspray will cause damage to the surfaces if not immediately wiped up. MTI recommends hand towels or wipes to avoid the possibility of overspray.

MTI has tested a number of premixed disinfecting solutions for use on all surfaces of our products. However, all disinfectants and cleaning solutions will wear and eventually damage material to some extent. MTI provides these recommendations to assure our customers receive the longest product life possible.

The following are believed to be safe for these surfaces:

Mfg	SKU	Description	Unit Size	CA/QTY EA
<b>PDI</b>	P63884	Sani-Cloth® AF3 - X-Large Canister	7.5" x 15"	6/65
<b>PDI</b>	P13872	Sani-Cloth® AF3 - Large Canister	6" x 6.75"	12/160
<b>PDI</b>	U27500	Sani-Cloth® AF3 - X-Large Packets	11.5" x 11.75"	3/50
<b>PDI</b>	H59200	Sani-Cloth® AF3 - Large Packets	5" x 8"	10/50
<b>Clorox Healthcare</b>	30824	Hydrogen Peroxide Disinfectant Wipes	6.75" x 9"	6/95
<b>Clorox Healthcare</b>	30825	Hydrogen Peroxide Disinfectant Wipes	6.75" x 5.75"	6/155
<b>Clorox Healthcare</b>	7105	EZ-Kill® Disinfecting Cleaning-Wipes	10" x 10"	12/65
<b>Clorox Healthcare</b>	7110	EZ-Kill® Disinfecting Cleaning-Wipes	6" x 6.75"	12/160
<b>Clorox Healthcare</b>	35309*	Clorox® Germicidal Wipe*	6.75" x 9"	6/70
<b>Clorox Healthcare</b>	30577*	Clorox® Germicidal Wipe*	6" x 5"	6/150

\*Overtime, bleach will leave a white residue.

These products are readily available through your medical/surgical healthcare distributors, either in single wrapped packets or more economical canisters.

Disinfectant manufacturer contact information:



Toll Free: (800) 999-6423  
 Phone: (845) 365-1700  
 e-mail: info@pdipdi.com  
 www.pdipdi.com



Toll Free: (800) 638-2625  
 Phone: (904) 996-7758  
 e-mail: custsvc@hlk.cc  
 www.hlk.cc